



Designed to Facilitate Maximum
Bone Volume Retention.



Endobon[®] Xenograft Granules

Regeneration
by Design



Endobon Xenograft Granules

Designed to Facilitate Maximum Bone Volume Retention



Endobon Xenograft Granules adhere to one another when hydrated for easy transfer to the defect site.



Small Granules

500–1000 μm particle size preferred for grafting smaller defects, such as extraction sockets.



Large Granules


1000–2000 μm particle size preferred for grafting large defects, such as sinus elevations.

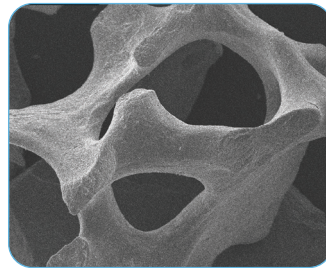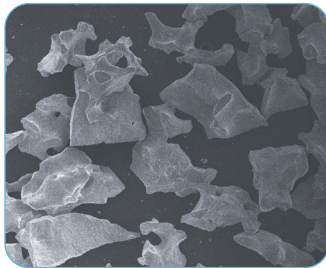

SEM images of Endobon Xenograft Granules at 20x and 100x showing the micro and macro pores in the particles.

Conveniently Packaged in Easy-to-Open Dishes.

Value Packs (5 ml and 8 ml) are individually packaged in 1 ml containers for single-unit dosing.

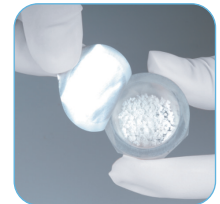

A Bone Graft Substitute

- Bovine-derived hydroxyapatite that has been fully deproteinized by a two-step, high temperature process for protection from bacteria, viruses, and prions
- An essentially non-resorbable material that is ideally suited for regeneration of bone defects when effective space maintenance is required
- Osseoconductive due to the interconnecting micro and macro pores for bony integration, which facilitate graft stability and vascular ingrowth¹

Indications

Endobon Xenograft Granules are indicated for the following dental and/or oral surgical procedures:

- Alveolar ridge augmentation/reconstruction
- Filling a socket after tooth extraction
- Sinus elevation

1. Hing KA, Best SM, Bonfield W. Characterization of porous hydroxyapatite. J Mater Sci Mater Med. 1999 Mar;10(3):135-45.

Anterior Ridge Augmentation

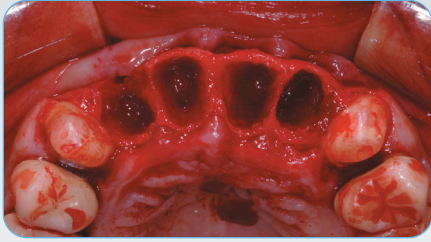


Fig. 1. Extraction sockets of the four maxillary incisors and immediate implant placement.

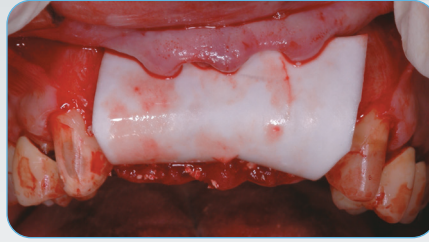


Fig. 2. Grafting with Endobon Xenograft Small Granules covered by an OsseoGuard® Resorbable Collagen Membrane.



Fig. 3. The soft-tissue flaps were closed and sutured.



Fig. 4. Clinical appearance of soft tissue showing excellent soft-tissue healing after four months.

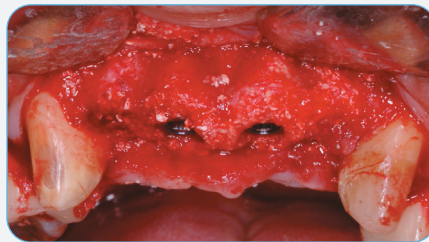


Fig. 5. Clinical appearance of the regenerated site at four months, after removing the remnants of the membrane.

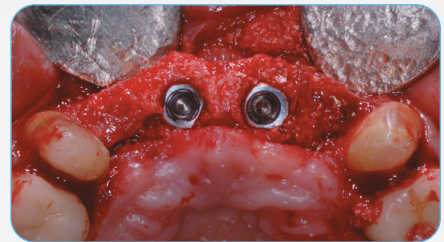


Fig. 6. Occlusal view after four months.

Post-Extraction Defects in the Aesthetic Zone



Fig. 7. Post-extraction defects in the maxilla right central and lateral incisor area.



Fig. 8. Occlusal view of the extraction site defects.



Fig. 9. Facial view of dehiscence defects after implant placement.



Fig. 10. Occlusal view of implants and defects.



Fig. 11. Grafting with Endobon Xenograft Small Granules covered by an OsseoGuard Resorbable Collagen Membrane.

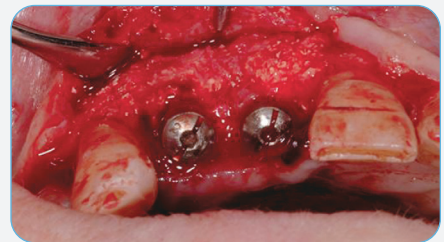


Fig. 12. Regeneration at four months after removing the remnants of the membrane.

Two Year Histology

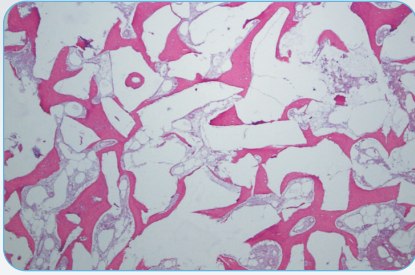


Fig. 1. Histologic core of the regenerated bone showing that the Endobon Xenograft Granules integrated with newly formed bone.

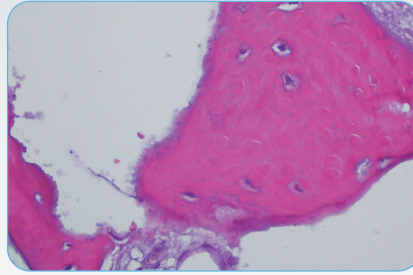


Fig. 2. At 40x magnification, note the newly formed bone (violet) and mature bone (pink) that surround the Endobon Xenograft Granules (white).

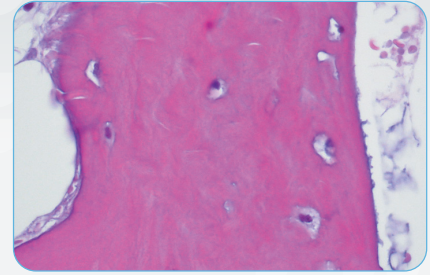


Fig. 3. At 60x magnification, lamellar bone is well formed.

Ordering Information

Endobon Xenograft Granules

Catalog #	Volume	Particle Size
Small Granules		
ROX05	0.5 ml	500–1000 μm
ROX10	1.0 ml	500–1000 μm
ROX20	2.0 ml	500–1000 μm
Large Granules		
ROXLG20	2.0 ml	1000–2000 μm
ROXLG50	5.0 ml*	1000–2000 μm
ROXLG80	8.0 ml*	1000–2000 μm

*Indicates a value pack. Value packs consist of multiple units of 1.0 ml each (5 units or 8 units respectively).



For more information, visit ZimVie.com

ZimVie
 4555 Riverside Drive
 Palm Beach Gardens, FL 33410
 1-800-342-5454
 Phone: +1-561-776-6700
 Fax: +1-561-776-1272

ZimVie
 Restoring Daily Life.®

Unless otherwise indicated, as referenced herein, all trademarks are the property of ZimVie; and all products are manufactured by one or more of the dental subsidiaries of ZimVie, Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. Endobon is manufactured by Biomet France, Sarl. Endobon and OsseoGuard Resorbable Collagen Membrane are distributed by ZimVie Dental and its authorized marketing partners. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie. ZV0970 REV A 05/23 ©2023 ZimVie. All rights reserved.

