


Driva™ Drills Usage Chart



Trabecular Metal™, TSX™, TSV® and Eztetic®


Use a dry erase marker to track the total number of osteotomy preparations that are performed with each drill. It is recommended that reusable drills be replaced after 15 uses. *Single use drills should be disposed of after each procedure.

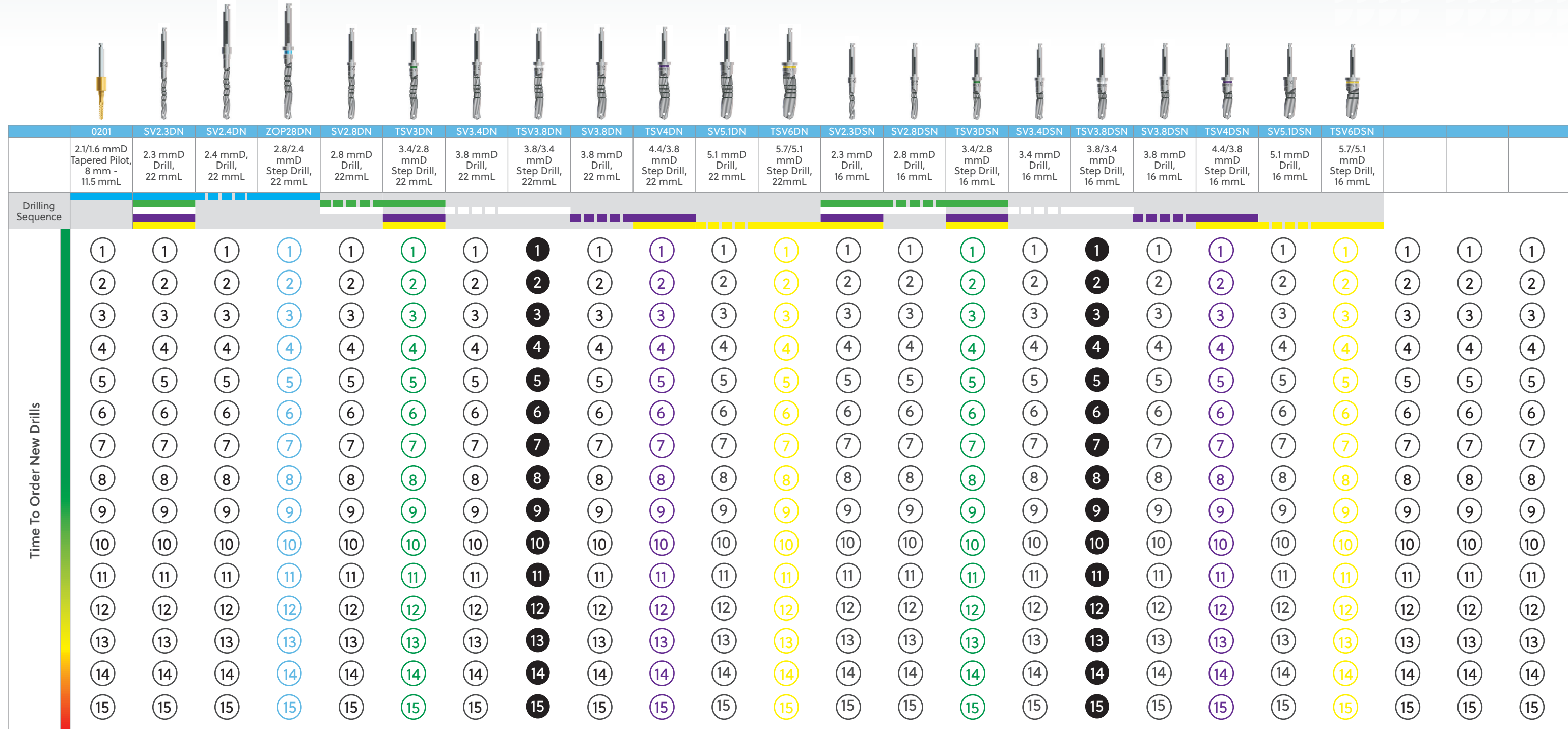

*Single use drills must be disposed of after each procedure. This chart is intended to be used as a reference guide only. The number of uses per drill will vary and depends on a variety of factors including bone density encountered, proper handling and cleaning. Refer to Instructions for Use (IFU) for Biomet 3i Kits and Instruments (P-ZBDINSTRP) available at labeling.zimvie.com for complete Cleaning and Sterilization Information.

NOTE: Surgical instruments are susceptible to damage for a variety of reasons including but not limited to: prolonged use, misuse, rough or improper handling, inadequate cleaning, etc. Care must be taken to avoid compromising the intended performance of the instrument.

Know how to identify excessive wear

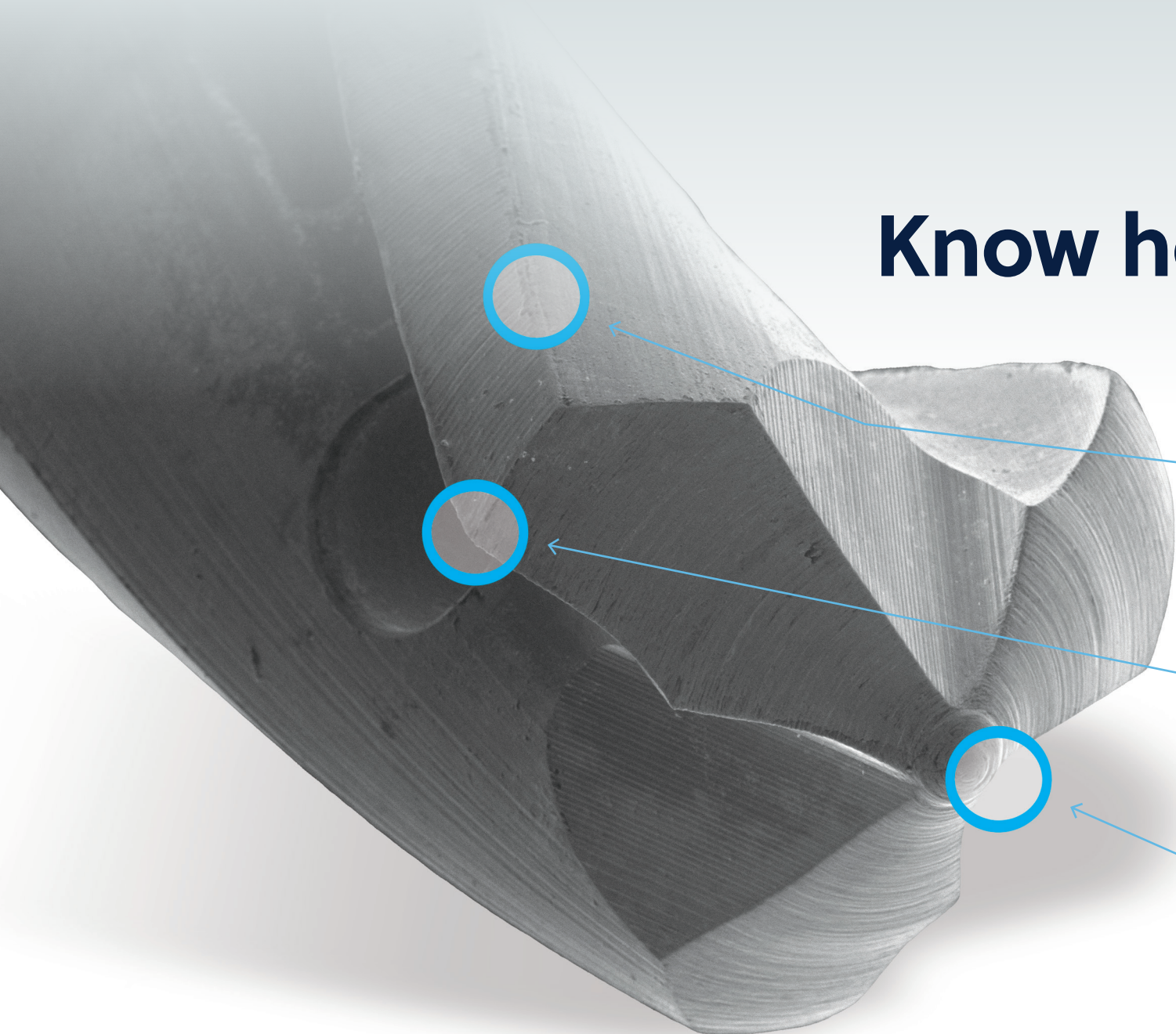

Dull Cutting Edge

Rounded Tip

Rounded Corner

Inspection for Wear and Damage

Before each use, visually inspect all devices and trays/kits for completeness, damage and/or excessive wear (e.g. corrosion or rust build-up on the instrument surface, structural wear or damage, grommet presence or damage partial or complete fracture. If damage or wear is noted that may compromise the function of the instrument or tray, DO NOT USE. Contact your ZimVie representative for replacement.

Refer to Instructions for Use (IFU) for Biomet 3i Kits and Instruments (P-ZBDINSTRP) available at labeling.zimvie.com for complete Cleaning and Sterilization Information.

ZimVie Dental Global Headquarters
4555 Riverside Drive,
Palm Beach Gardens, FL 33410

Contact us at 1-800-342-5454
or visit zimvie.com

Unless otherwise indicated, as referenced herein, all trademarks are the property of ZimVie; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Holdings, Inc. and marketed and distributed by ZimVie Dental and its authorized marketing partners. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie Dental. ZV1028 REV A 05/23 ©2023 ZimVie. All rights reserved.

